KIKSAFE[®]

MOTO FIRST AID KIT

Kit Pronto Soccorso per Moto e Bici

Instruction for immediate life-saving measures in case of accidents

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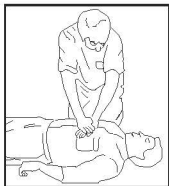
Specification contents DIN 13167

Indice dei contenuti DIN 13167

A) Giving help and reassurance, emergency call

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It is a moral obligation of every human being to help victims of accidents. Give first aid and care as good as possible.

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Have the rescue services called out as soon as possible.

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B) How to call for help

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Emergency call

Where? Give exact details about the place of emergency!

What happened? Give a short description of the accident scene!

What number? Tell how many persons are injured. What kind of injuries!

What kind of injuries? Specify if there are any threatening injuries!

Wait! The operator will finish the call when he/she has all information he/she needs!

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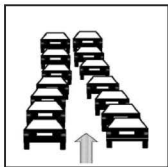
C) How to act at the site of an accident

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Allow ambulances to pass between the waiting vehicles.

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First of all safeguard the site of the accident, put up warning triangle.

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Assess the situation and act cautiously and considerately.

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Address casualty, open the safety belt and grasping his/her hips turn casualty's back towards you.

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Get ahold of casualty's forearm passing your arms under his/her armpits.

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Drag casualty out of the vehicle and put him/her down in a safe place.

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E) Removing a crash helmet

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**Open the visor and address casualty.
If not responsive: (G) Recovery position.**

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**Remove crash helmet by two first aiders
as shown here.**

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**Check breathing. If casualty is breathing:
(G) Recovery position.**

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F) Checking the vital functions

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**Patient does not respond:
(G) Unconscious.**

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**No breath to be heard, no movement of the
chest: (H) Respiratory failure.**

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**No signs of life like breathing, movement,
swallowing, coughing: (I) Circulatory failure.**

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**If not responsive, check breathing:
If breathing is present recovery position.**

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**Stretch near side arm and push it
under the hip.**

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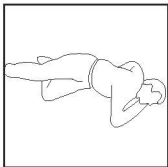
Get ahold of far side shoulder and hip and turn casualty cautiously on his/her side.

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Recline his/her head, open his/her mouth, and bend backward arm.

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Keep casualty warm and call for the ambulance.

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Mouth-to-nose respiration

In case of respiratory failure:

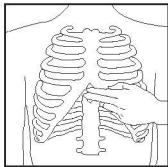
1. Recline head.
2. Keep nose or mouth closed.
3. Give breath into mouth or nose.
4. Turn your head and watch for breath sign.
5. Give 15 breaths per minute.

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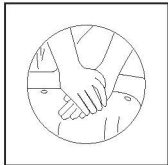
Cardio-pulmonary resuscitation (I + H)

Cardio-pulmonary resuscitation:

Continue CPR until a doctor is available, applying artificial breathing and chest compression (100/min) at a rate of 2:15.

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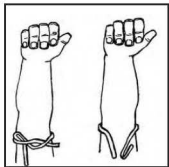
J) Bleedings

Sanguinamenti

In case of severe bleedings from the arm, first elevate arm and apply indirect pressure at the upper arm.

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Dress severe bleedings from arm or leg by applying a pressure bandage.

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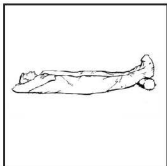
K) Shock

Scossa

Casualties who have a shock are pale, they are cold, and they have a feeble pulse.

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Place casualty with his/her legs in an elevated position, reassure and keep him/her warm. There is a rescue foil in the first aid kit.

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L) How to position a casualty

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In case of injuries or illnesses in the abdominal region, place casualty with bent knees for relaxation as shown here.

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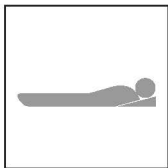
In case of breathing difficulties or cardiac infarct, place casualty with head and chest elevated if he/she is responsive.

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L) How to position a casualty

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In case of head injuries such as a concussion of the brain: place casualty with his/her head elevated if he/she is responsive.

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A casualty who has a shock is positioned on his/her back with his/her legs elevated if he/she is responsive.

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First Aid Dressing Bandage

Tampobenda sterile

Sterile wound dressing or compression bandage

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DIN 13151-G

First aid blanket

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To protect against heat and coldness

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Scissors

Forbici

To tear-off clothing and to cut the bandage material

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1

DIN 58279-A 145

Disposable Vinyl Examination Gloves

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2
4

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Cleansing Wipes

To remove dirtiness
or sterile wound

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Booklet giving
First aid
instructions

Manuale pronto
soccorso

1

KIKSAFE®